

Care and maintenance of the probe

- **Use a new ear tip for every patient.** Select an appropriately sized ear tip for the patient's ear canal – the ear tip should fit snugly. Use only the ear tips supplied with this probe.
- **Brush the probe every time after use with the cleaning tool supplied.** If cerumen or vernix is pushed up inside the probe housing, the probe will be damaged. Brush in a direction away from the probe housing, towards the tip of the probe.
- **The probe should never be placed directly in water or other cleaning liquids.** Liquids seeping into the housing will damage the probe.
- **When fitting a new ear tip to the probe, ensure it is pushed all the way to the probe casing.** There must be no gap between the ear tip and the casing. The top should be flush with the end of the probe tip.
- **The probe cannot be dismantled.** This is not necessary for cleaning if the care instructions are followed.
- **Do not fit the probe by holding on to the cable.** The cable could be damaged by doing this. The cable is not exchangeable.

MADSEN Capella



otometrics

MADSEN • AURICAL • ICS